Venus White *ultra*+

Whitening Trays

Make every day
a Venus White day.

INSTRUCTIONS FOR USE:

1

Carefully peel back foil and remove tray.

2

Place trays on upper, then lower teeth. Gently press tray with fingers to spread the gel around teeth.

3

Repeat once a day, for up to 7 days.

Keep trays in mouth for 15-20 minutes. Remove trays and rinse mouth with water to remove any remaining gel from teeth. Discard trays upon removal. Do not swallow.

IMPORTANT INFORMATION: Please read before use.

Caution:

- Keep out of reach of children.
- Avoid swallowing the cosmetic or part thereof.
- Avoid contact of the product with the eye.
- Consult your dentist before use if you have sensitive teeth or sore gums.
- Some people may experience temporary gum sensitivity when using Venus White Ultra+.
- If irritation (such as redness, swelling, soreness) of the gums or the mouth occurs, discontinue use and consult a dentist.
- Use for periods of longer than 14 days is to be only under the supervision of a dentist.
- Avoid direct contact of the active surface of the tooth whitening product with the gums and/or salivary flow.
- Do not eat, drink or smoke while trays are in mouth.
- Avoid contact with fabrics as gel may alter the color.
- This product will whiten only natural teeth.
- Women who are pregnant or breastfeeding should not bleach.

Do not use:

- unless you are age 12 or older.
- If you are allergic to any ingredient in this product.
- If you have dental braces.
- When sleeping.

FREQUENTLY ASKED QUESTIONS

How do Venus White Ultra+ Whitening Trays work?

Venus White Ultra+ Whitening Trays contain a controlled amount of advanced mint-flavored whitening gel. The signature trays, which are thin and flexible, are designed to fit each individual tooth and provide uniform coverage, whitening tight spaces even between teeth. The whitening gel works beneath the tooth's surface to erase stains and visibly brighten teeth. For maximum results, use for 7 days.

Is using Venus White Ultra+ Whitening Trays safe for my tooth enamel?

Venus White Ultra+ Whitening Trays are a safe and effective way to brighten your smile. They contain the same type of whitening agent used by dentists to whiten teeth quickly and safely. Use as directed.

What if I have dental work?

Venus White Ultra+ Whitening Trays will only whiten natural teeth. It will not whiten fillings, crowns, and other restorations. Consult your dentist before beginning treatment if you've had any of the above or if you experienced tooth staining caused by medication.

What if I experience tooth and gum sensitivity?

Some people may experience short-term tooth or gum sensitivity when using whitening products. If you experience sensitivity, we suggest the following:

- Do not brush your teeth immediately prior to usage.
- Try wearing trays for a shorter period of time; if sensitivity goes away then resume normal usage.
- Use in conjunction with a sensitive toothpaste before and during whitening treatment.
- If tooth sensitivity or gum irritation persists, discontinue use and consult a dentist.


Sometimes I see white spots after use. Will they go away?

Some people may experience white spots after initial use. The spots are temporary and will go away with continued use.

How often should I use Venus White Ultra+ Whitening Trays?

To keep your smile looking its absolute best and whitest, we recommend that you repeat the Venus White Ultra+ Whitening Trays treatment every six months or as needed.

Thank you for joining us in our commitment to support the fight against breast cancer. Kulzer donates a percentage of every sale of Venus White Ultra+ to support breast cancer research.

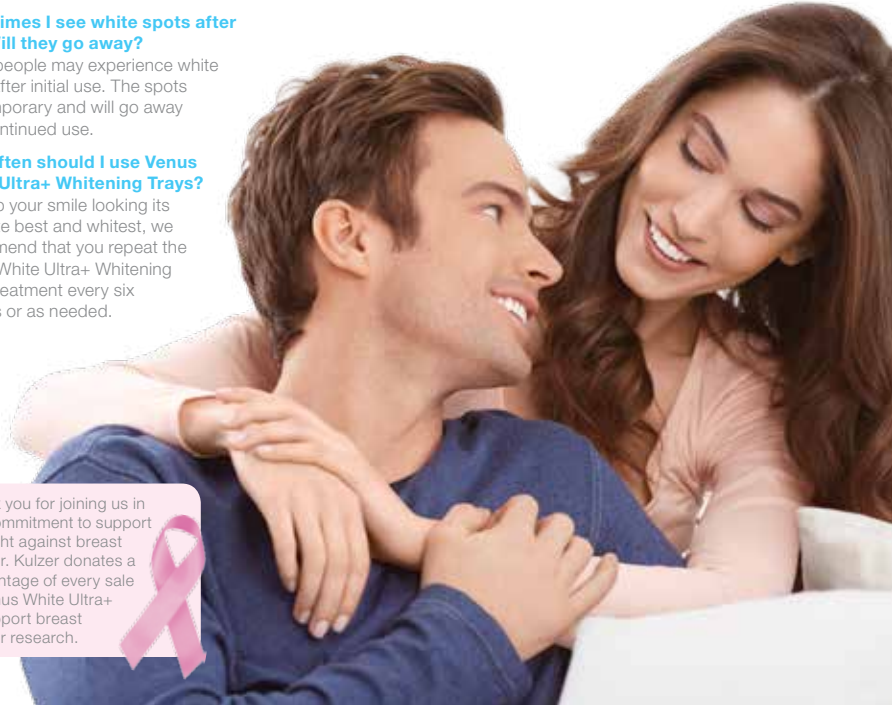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