**Description:**

Venus White Max In-Office Teeth Whitening System is a 38% hydrogen peroxide power bleaching gel. The unique syringe mixing system ensures freshness for each application. One syringe contains a chemical activator. The other syringe contains concentrated hydrogen peroxide. When mixed, Venus White Max is ready for use.

Gingival Barrier is a reflective, passively adhesive (sealing) light-cured resin. It is designed with sufficient strength to maintain a barrier yet is easily removed from embrasures and undercuts.

**Indications:**

Venus White Max is for in-office use only. It is used for dentist-applied bleaching of one or more teeth, parts of a tooth, and/or accelerated chairside whitening technique. Venus White Max is also used on non-vital teeth, including intracoronal bleaching.

Because restorative materials will not whiten, we recommend whitening teeth before esthetic restorative placement (wait two weeks after bleaching procedures before placing adhesive restorations). Use Venus White Max alone or in conjunction with the dentist-supervised Venus White Pro take-home whitening system or Venus White Ultra pre-filled disposable whitening trays. Our experience shows that most treated teeth continue to whiten 12-24 hours after treatment. Our Gingival Barrier is used to protect soft tissue adjacent to the teeth during in-office bleaching, microabrasion, etc.

**Preliminary Procedures:**

1. Remove any calculus or extrinsic stains before the bleaching appointment. This is ideally done at least one week prior to the bleaching appointment to prevent sulcular seepage from compromising Gingival Barrier placement.
2. Determine and record shade before bleaching.

**WARNING: Shield patient's and clinician's eyes with protective eyewear with side shields during mixing and applying.**

**Isolation:**

Isolate teeth to be bleached with our Gingival Barrier light-cured resin (Fig. 1) or with conventional rubber dam.

**Isolation Procedure Using Gingival Barrier:**

1. Place self-supporting lip/cheek retractors.
2. Remove Luer lock cap and attach the plastic 20ga bent mini delivery tip securely onto the Gingival Barrier syringe.

3. Completely rinse and air dry teeth, paying special attention to the gingival third.

Note: When isolating the mandibular arch, prevent saliva from flowing through embrasures of anterior teeth by using saliva ejector and cotton rolls in the sublingual region.

**Fig. 1** Express Gingival Barrier resin to build a 4-6mm by 1.5-2mm thick strip onto gingiva. Lap approximately 0.5mm onto enamel.

4. Lapping approximately 0.5mm onto enamel, begin building a resin barrier by expressing Gingival Barrier resin along the gingival margin 1.5 - 2mm thick. Continue building the barrier apically until approximately 6mm of the tissue is covered. Extend resin beyond the last tooth to be bleached. Where open embrasure(s) exist, express resin through the opening onto lingual, filling embrasure space completely.
5. After resin barrier is applied, cure resin by scanning light over surface for 20-30 seconds (Fig. 2). Clean teeth with prophy paste to remove any excess resin film. Use caution not to dislodge cured Gingival Barrier.
6. Place a tongue/bite block ( Fig. 3) in molar region and instruct patient to hold it in place with gentle but firm biting pressure.

*Fig. 2 Cure resin with a quality curing light 20 seconds.*

*Fig. 3 Place bite block*

#### **Alternative Isolation Procedure Using Rubber Dam:**

1. For safety, ligate rubber dam clamps with floss. Place appropriate clamps, bilaterally, one tooth posterior to teeth being whitened.
2. Punch holes and position rubber dam over clamps and teeth.
3. Attach rubber dam to frame. Using a rubber dam instrument and air, invert rubber dam border around teeth for maximum tooth exposure and optimum rubber dam seal. Check rubber dam for tears or areas of possible leakage.

#### **Application of Venus White Max:**

1. Allow contents of syringe to come to room temperature before mixing the two chemicals together. **WARNING: Do not mix chemicals when cold.** To activate, press chemical from the one syringe to the other. Reverse action and mix a minimum of 25 times rapidly (12-13 each direction/side) (Fig. 4). Press all mixed chemical into syringe labeled "Whitening Gel." Remove and dispose of empty "Activator" syringe.

*Fig. 4 Activate by pressing plunger 12-13 times each direction/ side. Finish with total contents in "Whitening Gel" syringe. Remove "Activator" syringe and dispose. Twist tip firmly onto "Whitening Gel" syringe.*

Securely attach the Flocked 22ga metal delivery tip onto syringe labeled "Whitening Gel."

NOTE: Once mixed, Venus White Max expires after ten (10) days. Keep refrigerated.

2. Check material flow on a cotton gauze or mixing pad to ensure evenness of flow before using intraorally. If resistance is felt, DO NOT proceed. Attach a new tip and test again before application.
3. Apply a 0.5 to 1.0mm thick layer of bleaching gel to the labial surfaces and extend slightly onto incisal/occlusal edges (Fig.5).
4. Each application typically lasts between 10-20 minutes. For optimum effectiveness stir/agitate every 5 minutes.
5. To remove bleaching gel from teeth, use a surgical-type suction prior to rinsing. This will remove the bleaching gel without splattering onto the soft tissue. DO NOT use water during initial suction. Use caution not

*Fig. 5 Apply a 0.5 to 1.0mm thick layer of mixed chemical. Place the gel on the labial surfaces. Extend slightly onto incisal/occlusal edge.*

to disturb or dislodge the resin barrier (Fig. 6). After all visible gel is removed, follow with a gentle rinse, directing water towards incisal edge. Place the high-volume suction at half-volume near the incisal edge to evacuate water. Gently air dry teeth. *Always use caution to not dislodge resin barrier.*

**Fig. 6** Remove whitening gel using suction using a surgical aspiration tip.

6. Reapply Venus White Max (Steps 3-5) three to four times, or until desired results are achieved. Do not exceed six applications in one visit.
7. Evaluate the shade change after each application. The total procedure lasts approximately 60 minutes. If additional whitening is desired, and no significant tooth sensitivity is observed, schedule patient for repeat treatment at least 3 days following the treatment. Alternatively, follow up with Venus White Pro take-home system with custom trays or Venus White Ultra prefilled, disposable whitening trays.

### Clean-up:

#### 1. For Gingival Barrier:

- a. After thoroughly suctioning off whitening gel (Fig. 6), rinse teeth with air/water syringe and vacuum (Fig. 7). Lift Gingival Barrier from surface using an explorer or instrument. All or large pieces will usually lift off at one time (Fig. 8).
- b. Remove Gingival Barrier interproximally using explorer and/or floss. Rinse interproximally with firm air/water spray.

**Fig. 7** Rinse thoroughly directing water toward high-volume suction.

#### 2. For Rubber Dam:

- a. After thoroughly suctioning off whitening gel, rinse teeth with air/water syringe and vacuum. Remove clamps and rubber dam.

**Fig. 8** Lift Resin barrier from surface using an explorer or instrument.

### Notes:

1. Venus White Max may be used in conjunction with the Venus White Pro take-home system or Venus White Ultra pre-filled disposable whitening trays. Usually 2-4 nights of treatment are sufficient. Wait 24 hours after in-office procedure before starting patient on dentist-supervised home bleaching materials.
2. Should patient experience post-op sensitivity, dentist can prescribe Venus Comfort Gel in a tray for one hour per day until condition subsides.
3. Do not etch teeth prior to bleaching.

### Nonvital Teeth:

**WARNING:** When placing ANY intracoronal whitening agents, confirm coronal seal prior to placement. DO NOT place product below the alveolar crestal bone.

1. Check radiograph for alveolar crest height.
2. Open lingual access to pulp chamber and remove all filling material, taking care not to dislodge gutta percha.
3. Place a 1-2mm glass ionomer filling on pulpal floor at or coronally to the alveolar crest height. Filling material should be thick enough to completely seal the obturation orifices and material.
4. Follow "Isolation" instructions to protect tissue. If gingival barrier is used, apply the gingival barrier to both the labial and lingual gingiva of the nonvital tooth and the teeth directly adjacent to the treated tooth.

5. Prepare bleaching gel and syringe as instructed in previous section. Verify flow prior to applying intraorally.
6. Express bleaching gel into the opened pulp chamber and onto the labial surface. After 10 minutes, remove gel from tooth using a surgical-type suction. Follow with a thorough rinse and high-volume suction at half volume. Air dry teeth. *Use caution not to dislodge gingival barrier.*
7. Repeat until desired results are achieved. If significant results are not achieved in 20-30 minutes (total time per tooth), discontinue.
8. When desired color is reached, follow “Clean-up” procedure as previously instructed.
9. Wait 2 weeks before final bonding or placing esthetic restoration. Note: Regression of whitening should be expected. Compensate by slightly over bleaching the tooth. Understand each patient’s expectation before beginning any whitening procedure.

### Single- or Partial-Tooth Bleaching:

1. The recommended barrier technique for single tooth bleaching is “Isolation Procedure Using Gingival Barrier.” The rubber dam technique may be used if desired.
2. Apply Venus White Max to discolored or stained tooth (or part of tooth), according to directions above.
3. Repeat procedure until desired results are achieved.
4. Follow “Clean-up” instructions for removal.

### Warnings/Precautions:

1. **WARNING: Clinician and patient must wear protective eyewear with side shields during entire procedure!**
2. Do not mix chemicals in syringe when cold. The product is thicker when cold and the force required to mix can break the syringe.
3. Check material flow on an inert surface (i.e. mixing pad) to ensure evenness of flow before using intraorally. If resistance is felt, DO NOT proceed. Attach a new tip and test again before using on patient.
4. Venus White Max MUST be refrigerated. Shelf life is 18 months refrigerated. Once mixed, Venus White Max is good for 10 days. Keep refrigerated. After 10 days, discard using proper disposal methods, as syringe may develop pressure. Materials should be diluted with water and disposed down a drain or toilet followed by a few liters/quarts of additional water. DO NOT discard in garbage cans containing burnable material such as paper, cardboard, rubber, leather, cotton, and the like, as spontaneous combustion can result. Do not bring gel into contact with materials listed previously during application as combustion may occur.
5. Wear protective gloves and eye covering when handling this product! Hydrogen peroxide is a strong oxidizing agent and can cause burns to soft tissues (i.e. gingiva/lips). Keep isolated to area of treatment. Vacuum gel off teeth before rinsing with water to prevent splattering.
6. Gingival and general oral health should be confirmed before treatment.
7. Although there is no evidence to the contrary, because of safety precautions, DO NOT use Venus White Max on patients who are pregnant.
8. Venus White Max is contraindicated for patients with known allergy or chemical sensitivity to peroxides or glycols.
9. Restorations should be water-tight, and all exposed sensitive dentin should be covered. If areas are sensitive or potentially so, treat with proper restoration, dentin bonding agent, etc.

10. Larger recurring bubbles in bleaching gel may indicate leakage of peroxide gel onto gingiva or presence of organic debris. Vacuum off gel, wash, dry, and inspect.
11. Areas of hypocalcification, clinically not visible, occasionally exist. Because of mineralization differences, these areas will whiten faster than normal enamel and will become more visible during bleaching. Continue bleaching until remaining tooth surface more closely blends with hypocalcified area (additional blending usually occurs later during normal rehydration following completion of bleaching). Additional appointments and/or dentist-supervised tray bleaching may be required. Re-evaluate in two weeks when color has stabilized.
12. Determine and understand patients' expectations before beginning treatment. Inform patients that existing restorations, crowns, and bridges do not lighten. Inform patients that because of unforeseen factors or enamel variations, some teeth may not meet desired expectations.
13. Caution should be used to prevent transfer of bleaching gel to adjacent tissue(s) from inadvertent contact from gloved finger, vacuum tip(s), etc. Affected tissues should immediately be rinsed thoroughly clean with water.
14. Clinician should wait two weeks following the bleaching procedure before matching restorations to tooth shade to ensure stabilization of color and to ensure all peroxide has leached from the enamel and/or dentin. Residual peroxide (oxygen) in the tooth will prevent adequate bonding.
15. Monitor patients' bleaching progress during procedure and with take-home gel to prevent over-bleaching and/or excess enamel translucency.
16. Some enamel stains do NOT respond to bleaching. Inform your patient of this possibility before treatment. A microabrasion slurry product may be used on surface stains that do not bleach.
17. Venus White Max is supplied in preloaded syringes. Material is NOT to be injected. When empty, dispose of syringes properly. Avoid disposing of syringes in receptacles containing combustible products. Strong peroxides can initiate combustion.
18. Do not use Gingival Barrier on patients with known sensitivity to resins. If dermatitis, rash or any other allergic reaction develops, wash area with water and refer patient to a physician.
19. Do not anesthetize teeth for in-office bleaching.

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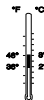
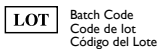
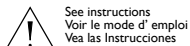
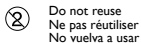
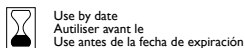
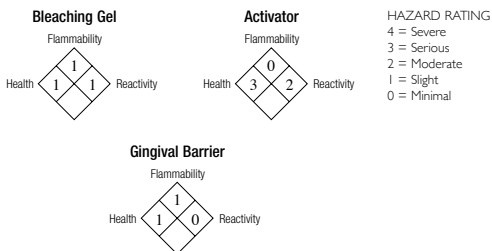
Venus White Max syringes have an expiration date stamped on them consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year. Venus White Max syringes can also have the actual expiration date and lot number printed on them.

U.S. federal law restricts the sale of this device by or on the order of a dentist.  
Keep out of reach of children.

U.S. Patent Numbers: 5,858,332, 6,503,485, 6,746,664, 5,770,182, 5,759,038,  
5,376,006, 5,098,303, 5,409,631, 5,746,598, 5,725,843, 5,985,249, 5,855,870,  
5,851,512, 6,036,943, RE38,721, RE38,823, RE38,722.

Distributed by:

Kulzer, LLC  
4315 South Lafayette Blvd.  
South Bend, IN 46614-2517  
1-800-431-1785

Made in USA

91087.4 090513

**KULZER**  
MITSUI CHEMICALS GROUP