## Description:

Venus Comfort Gel is a unique, sustained release, 3% potassium nitrate and 0.11% weight fluoride ion, sticky, viscous teeth desensitizing gel. Venus Comfort Gel is designed to be used in a custom-fabricated tray.

## Indications:

Venus Comfort Gel provides a film-like varnish for sensitive teeth, sealing dentinal tubules of exposed dentin and other areas where post-operative sensitivity is a concern. The product is used either by a dental professional in the office or provided to the patient for home treatment of dentin sensitivity.

## Procedure:

1. Read and understand instructions before using or dispensing to patient.
2. Fabricate tray to meet patient's needs. Areas of sensitivity should be indicated on cast before sending the model. If patient already has a custom bleaching tray, Venus Comfort Gel maybe be applied in the same tray and manner as bleaching gel.
3. Venus Comfort Gel is dispensed in 1.2ml unit dose syringes. A complete arch should use most of the contents of a syringe. If sensitivity is localized, place gel in affected area(s) and a small amount in the molar region to hold tray in place.
4. The length and number of times tray is worn depends on condition, patient, and clinician. Recommended treatment times can range from 15 minutes to 1 hour.
5. Tray should be removed at meal times. Immediately following tray removal, patient should brush and rinse teeth with water to remove all remaining gel. DO NOT swallow solution during rinsing. Wash tray and store in a tray storage case. Fresh gel should be used with each application.
6. Gel may be warmed before placing in tray to avoid thermal sensitivity.

**Precautions:**

1. Keep desensitization product away from small children.
2. This product is NOT for nighttime use.
3. Do not use tobacco products or eat while using this product.
4. Not intended for use by pregnant women.
5. If any problems occur, discontinue treatment and notify your dentist.
6. Keep out of heat and/or sunlight at all times. Venus Comfort Gel should be stored at room temperature. Shelf life is 36 months.
7. Patients should not swallow large volumes of Venus Comfort Gel. Excess material at insertion and residue upon removal should be brushed off and rinsed: DO NOT swallow.

**General Precautions:**

1. For Professional use only.
2. Review instructions, precautions, and MSDS before beginning treatment.  
Use only as directed.
3. Confirm that patient has no known allergies to treatment materials.

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

Kulzer syringes have an expiration date stamped on them consisting of one letter and three numbers. The letter is a lot number used for manufacturing purposes and the three numbers are the expiration date. The first two numbers are the month, and the third number is the last number of the year. Kulzer syringes can also have the actual expiration date and lot number printed on them.


## Venus Comfort Gel

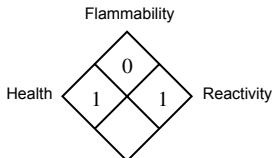## Gel Confort Vénus
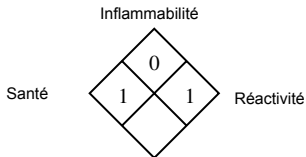
## Venus Comfort Gel
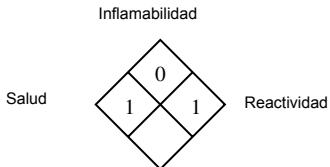
### HAZARD RATING

- 4 = Severe
- 3 = Serious
- 2 = Moderate
- 1 = Slight
- 0 = Minimal

U.S. federal law restricts the sale of this device by or on the order of a dentist.  
Keep out of reach of children.

Distributed by:

**KULZER**  
MITSUBISHI CHEMICALS GROUP

Kulzer, LLC.  
4315 S. Lafayette Blvd.  
South Bend, IN 46614  
(800) 431-1785  
Made in USA

17164.2 052418