Kulzer, LLC  
4315 South Lafayette Blvd.  
South Bend, IN 46614-2517  
1-800-431-1785 www.KulzerUS.com

Made in USA  
55370104 R/17

**KULZER**  
MITSUBISHI CHEMICALS GROUP

#### For use by dental professionals only.

Inspect and sterilize clamp before each use. For optimum use, use a general purpose cleaner prior to sterilization. Do not heat clamp above 350°F/177°C.

Choose the appropriate size and style clamp giving consideration to the tooth size, location, gingival architecture, and matrix needs. **Practice proper rubber dam technique at all times.** Use rubber dam punch to place holes in the rubber dam according to applicable dentition. **Attach floss to both of the clamp jaw forcep holes.** Mount the rubber dam to the rubber dam frame, and tie floss safety lines to the frame. Set the clamp into prepared rubber dam and place on tooth, checking the stability of the clamp before proceeding. Use additional floss to seat rubber dam between isolated teeth.

**Caution: Modification, over-extending, bending, or use exceeding one year may cause breakage. Do not place clamp in mouth until the rubber dam has been properly placed. Clamp could become a choking or safety hazard if dropped or broken in the mouth without proper use of the rubber dam at all times.**

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