

**PALA®**



## **Pala Denture Pass**

Giving a hand to oral health.



**KULZER**  
MITSUI CHEMICALS GROUP

## High Quality.

Congratulations on your new high quality denture.

These dentures were produced by a qualified oral health professional team according to the latest principles on function and design.



Patient



Date/Stamp

## Daily Denture care.

Always keep in mind

Even with the very best denture, you still have to allow for a period of adjustment. The palate, tongue and chewing muscles only adapt slowly to the new situation. Give yourself some time to “practice” chewing and speaking. Even if you are confronted with minor adjustment difficulties, you should continue to wear the denture. It may be helpful to start with easy-to-chew foods like fish or pasta. The following recommendations support you in maintaining the high quality of your denture over the long term:

- Visit your Oral Health Professional regularly
- Cleaning your dentures daily maintains their chewing function and also cares for your good looks
- Under no circumstances use aggressive liquids (eg. Cleaning detergents) for cleaning
- Never try to repair the denture yourself
- Follow the directions of your Oral Health Professional – they will be glad to advise you on all questions you have concerning your teeth and your dentures

## Why keep your dentures clean?

If you want to remain happy and content with your new dentures you must clean them carefully every day. Bacteria settle on dentures – as they do on natural teeth – this is called plaque, it not only looks ugly, but can lead to an inflammation of the soft tissue.

The formation of plaque on teeth or dentures can be reduced or completely stopped, but this depends entirely on the amount of care you give them.

Take especially care on the interdental space, the contact area to the gums and those parts which have contact to the residual teeth in mouth. Brush them thoroughly for some minutes. Don't forget to brush residual teeth, implants and any attachment parts in the mouth.



## How to clean correctly?

You can clean your dentures thoroughly with a denture brush or simply a soft tooth brush and non whitening tooth paste. Don't use household detergents like dish-washing or chlorine hygiene liquids. Nevertheless, it is important to use tooth paste in order to reach a sufficient cleaning. Prosthesis cleaning tabs might be a reasonable completion but do not replace classical tooth brushing. Clean your denture under running water after every meal. After you have removed your dentures, you should also rinse the inside of your mouth. If you want to keep your remaining natural teeth as long as possible, then you must clean them especially well – even if they have been replaced by crowns.

**Tip:** Always clean your dentures over a basin filled with water or lay a towel inside. In this way you avoid your dentures breaking if you should accidently drop them.



## Tips for patients with dentures.



Important:

Place your dentures in a glass with water as the denture base has to be kept moist.



Always remember:

The less teeth you can call your own, the more care should be taken for each one. The future of your teeth lies in your own hands!

## Think about your health.

Persistent difficulties necessitate an appointment with your Oral Health Professional or dental surgery in order to avoid deterioration.

Important – If you experience any of the following symptoms, please see your Oral Health Professional immediately.

### Formation of pressure sores

For various reasons ulcerations can form on the mucous membrane supporting the denture. These can lead to inflammation and infection. Persistent ulcerations can cause bone loss and therefore fitting problems.

### Fitting problems

Poor fit can cause damage or breakage of the denture. Even if this does not occur, the lost of retention can impair speech and lead to rattling of the denture.

## Plaque adhesion and discolouration

In the case of dentures worn for many years or those dentures subject to incorrect care, increased plaque adhesion may arise due to surface wear. The bad breath and damage to health caused by this can be avoided with suitable measures by your Oral Health Professional treatment team.

Extreme nutrition habits like continuous consuming of coffee, tea, coloured juices, tobacco etc. may discolour adhering plaque. Alcohol based solutions like mouth inhalers and washings may contribute. If discoloured plaque occurs, professional cleaning is recommended.



## Optional Teeth Warranty.

The high quality of Pala Premium, Mondial or Idealis teeth used in this denture are guaranteed for a period of 5 years. If within this period the teeth show any signs of excessive wear, breakage or discoloration, please contact your Oral Health Professional immediately.

This guarantee does not extend to the denture base material or the dental professional's fees.

## Used Materials

Pala tooth line	Anterior (Shade & Mould)	Posterior (Shade & Mould)	Denture base material

## Warranty content.

This warranty is extended to defects in the materials and workmanship of Kulzer teeth used in the fabrication of the denture. It does not extend to workmanship on the part of the laboratory or the dental practitioner. Kulzer will provide replacement teeth, provided that: (1) the prosthesis is a complete upper and lower denture; (2) all teeth in the denture are Kulzer teeth; (3) your personal warranty card is fully completed by your Oral Health Professional; (4) Kulzer teeth have not been misused in any way; (5) you have visited your dentist for regular check-ups.



Patient



Oral Health Professional's Signature and Date of Denture insertion

This warranty is restricted to the replacement of Kulzer artificial teeth only and does not cover reimbursement for any other costs incurred by patient, Oral Health Professional or dental laboratory.

All the best with your new denture!



**Contact in Germany**

Kulzer GmbH  
Leipziger Straße 2  
63450 Hanau, Germany  
info.lab@kulzer-dental.com